

RTU Concentrate Z-Sponge



For efficient manual pre-cleaning to remove gross contaminants from endoscopes and other soiled surgical equipment.

Instructions for use:

1. Check Z-Sponge to ensure that it is within its expiration date. (ex. 3112132, expires 12/31/2013)
2. Tear open package and add contents to approximately 3 – 5 gallons of water (70 – 120 °F). Squeeze sponge to ensure all enzymatic cleanser is added to the bath.
3. After use dispose of sponge in accordance with your facilities rules.
4. Rinse enzymatic cleanser from instruments and scopes prior to sterilization or high level disinfection.

Product Information:

- Contains Non-Foaming Multi-Enzymatic Cleanser
- Safe for all medical devices and scopes
- Non-Abrasive, neutral pH
- Single use sponge
- Non-foaming for optimal clarity

RTU Concentrate Z-Sponge

Reorder# **ZUTR50505**

Zutron Medical
2830 Roe Ln
Kansas City, KS 66103
877-343-5873
www.zutron.com

