


Concentrated Multi-Enzymatic Cleaners

Instructions For Use: Manual Cleaning



For breaking down bio-burden to facilitate the decontamination of soiled medical instruments, devices, and endoscopes.

1.

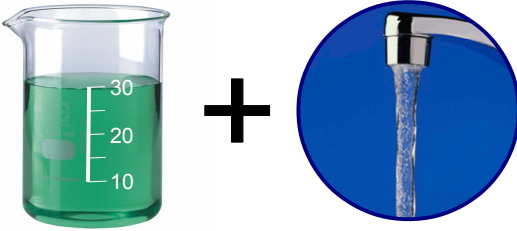

Dilute ½ oz – 1 oz of cleanser per 1 gallon of water.

2.




Soak instruments for a minimum of 2 minutes or longer as necessary.

3.



After soaking rinse bio-burden and cleanser from instruments.

4.

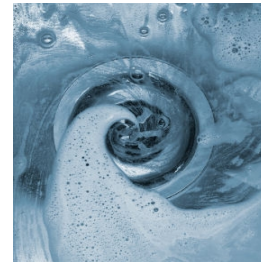

After each use dispose of and mix a fresh solution.

Cleanser information and precautions:

- Effectively breaks down blood, proteins, fat, carbohydrates, and synthetic liquids
- Safe for all surgical instruments, endoscopes and other general healthcare equipment
- Non-abrasive, neutral pH, and 100 % biodegradable formula
- Not to be used as a disinfectant
- Diluted cleanser must be rinsed from instruments prior to sterilization
- Do not use on instruments that will not be rinsed prior to sterilization
- Do not use at temperatures above 64°C
- Do not mix with other chemicals
- Do not allow cleanser to freeze