

RTU Foaming Z-Sponge



For efficient manual pre-cleaning to remove gross contaminants from endoscopes and other soiled surgical equipment.

Instructions for use:

1. Check Z-Sponge to ensure that it is within its expiration date. (ex. 3112132, expires 12/31/2013)
2. Tear open package and remove sponge. Ready for immediate use - do not dilute in water or mix with other cleansers.
3. After use dispose of sponge in accordance with your facilities rules.
4. Rinse enzymatic cleanser from instrument(s)/scope prior to sterilization or high level disinfection.

Product Information:

- Contains Multi-Enzymatic Foaming Cleanser RTU
- Safe for all medical devices and scopes
- Non-Abrasive, neutral pH
- Single use sponge

RTU Foaming Z-Sponge

Reorder# **ZUTR50504**

Zutron Medical
2830 Roe Ln
Kansas City, KS 66103
877-343-5873
www.zutron.com

