



# OPERATION MANUAL

## Table of Contents

Page 2	Important Safety Information
	1. Intended Use
	2. Safety
	3. Electrical Safety
Page 3	Getting to know the RTU Auto Foamer
Page 4	Getting started with your RTU Auto Foamer
	1. Charge the Sprayer Head
	2. Connect the suction tube
	3. Attach the Sprayer to the 30oz. Container
	4. Attaching the RTU Auto Foamer Tip System
Page 5	Troubleshooting and Maintenance
Page 6	Instructions for Use

## Important Safety Information

### Intended Use

The RTU Auto Foamer is for rapid application of Zutron's Multi-Enzymatic Foaming Cleanser RTU to medical instruments, equipment, and trays. Do not use this product for any other purpose.


### Safety

1. Read this entire manual before operating unit.
2. Do not use acids, caustics, or flammable liquids in the unit.
3. Always wear personal protective equipment when operating the RTU Auto Foamer.
4. Never point the spray wand at anyone.
5. Never place your mouth on the nozzle or suck nozzle to prime.
6. Do not use the Auto-Foamer if it has been damaged in anyway.
7. Do not attempt to disassemble the Auto Foamer, there are no user serviceable parts.
8. Avoid contact of Muti-Enzymatic Foaming Cleanser RTU with skin and eyes. If eye contact occurs, immediately flush eyes with water for 15 minutes and see MSDS sheet for other first aid measures.

### Electrical Safety

1. Do not handle charger or plug with wet hands, charge away from water sources.
2. Do not use with any other charger than the one supplied.
3. Never operate the sprayer when it is connected to the charger.
4. Do not expose the unit to flooding water.
5. Do not operate the unit if the cord is damaged or has been exposed to water.
6. Do not attempt to replace nickel-cadmium batteries, contact with metal or jewelry could cause burns.


## Getting to know the RTU Auto Foamer

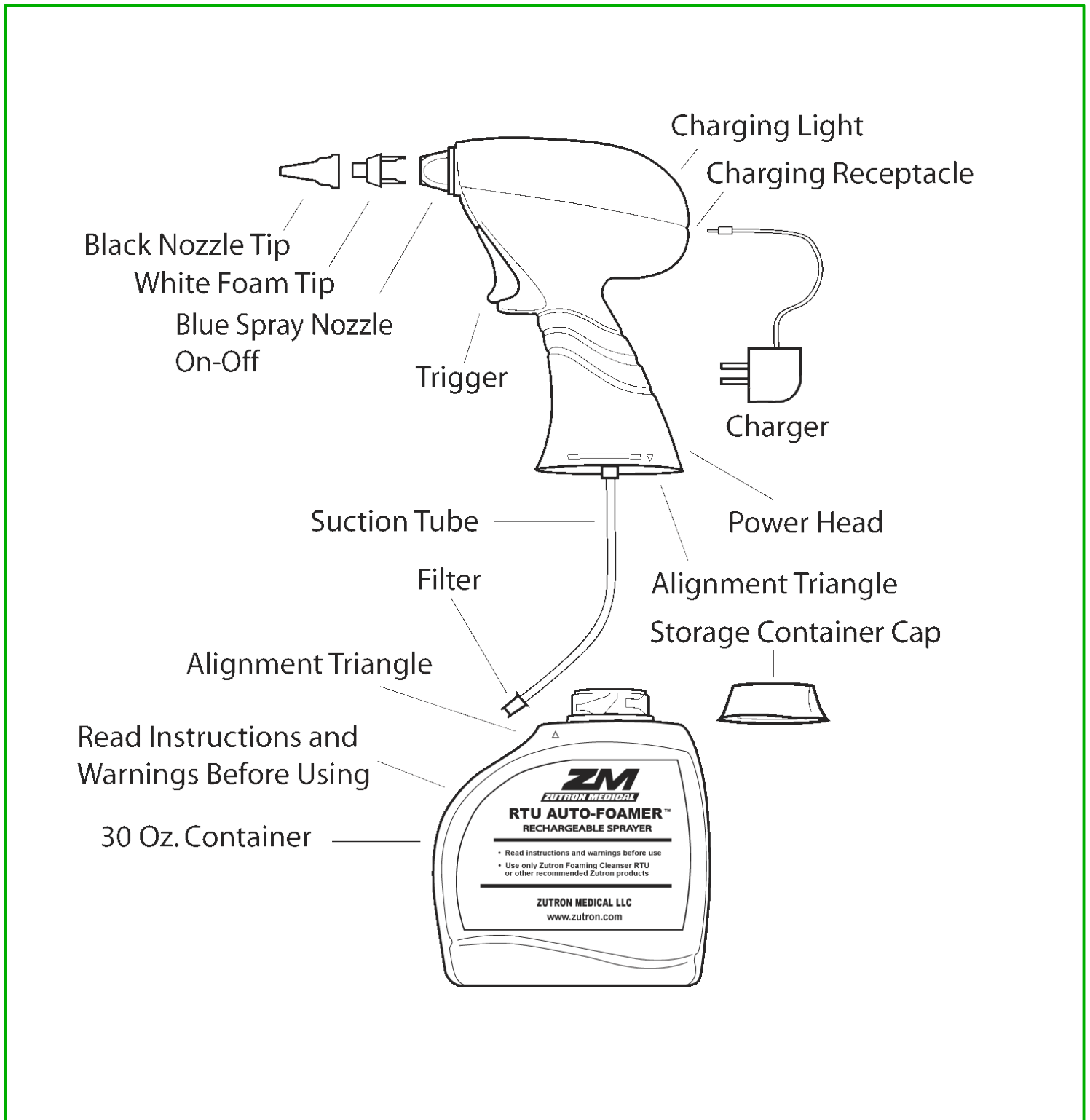

## Getting started with your RTU Auto Foamer


### 1. Charge the Sprayer Head

Connect the charger to the sprayer head. Plug the charger into a 110 v wall outlet. **Sprayer requires 12 hours of charging for a full charge.** Disconnect charger when complete.

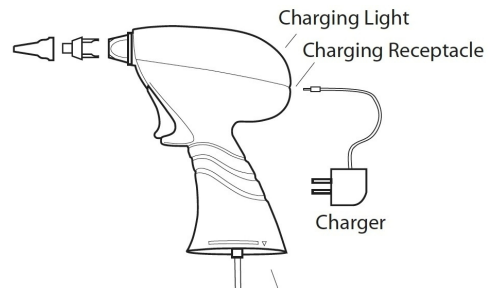


### 2. Connect the Suction Tube

Push the suction tube into the bottom of the sprayer head with the filter pointed down and in the same direction as the blue spray nozzle.

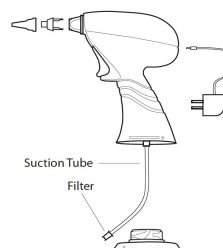

### 3. Attach the Sprayer to the 30 oz. Container

Align the triangle on sprayer head with the triangle on the 30 oz. container. Rotate the sprayer head ¼ turn clockwise. Unit should snap and lock securely closed.

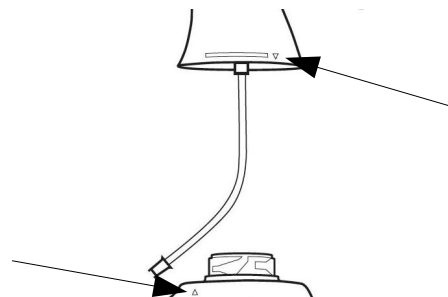

### 4. Attaching the RTU Auto Foamer Tip System

The RTU Auto Foamer Tip System allow application of foaming cleanser in a variety of spray patterns.

**BLUE SPRAY NOZZLE (ON-OFF)** – Sprays a fine mist.

**WHITE FOAMING TIP** – Attaches to the Blue Spray Nozzle with 4 prongs. To properly work there must be an 1/8" void between the Foam Tip and the Blue Spray Nozzle.

**BLACK NOZZLE TIP** – Pushes onto the Foaming Tip. Make sure the Foaming Tip is pushed all the way in. Ideal for spraying into instrument lumens.

